P

## BioAgePeel

Biorevitalizing

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### INSTRUCTIONS FOR USE

BioAgePeel

BioAgePeel is a medical device, a chemical peel containing trichloroacetic acid, mandelic acid and

kojic acid with an exfoliating action, specifically formulated to treat skin problems caused by acne

and skin dyschromia. Thanks to its exfoliating properties, the peel removes dead cells from the

stratum corneum and promotes cell turnover, thus indirectly stimulating the associated dermal and

epidermal regeneration in cell repair processes. BioAgePeel is therefore an ideal treatment to restore the skin to its optimal, natural condition.

Composition:

BioAgePeel: trichloroacetic acid (20%), mandelic acid, kojic acid, excipients.

Directions for use:

BioAgePeel

A chemical peel with a high concentration of trichloroacetic acid, which acts on the entire thickness of the epidermis, specifically designed to treat moderate to severe acne lesions (papules, pustules, nodules and cysts) and related scarring. This mix of acids provides intense exfoliation with a

depigmenting action to treat severe dyschromia. By removing the most damaged outer layers, cell

renewal is promoted and wrinkles, medium-depth and surface scars, and skin spots are reduced.

BioAgePeel is suitable for:

- Treating acne lesions and related scarring
- Treating skin dyschromia

BioAgePeel is a peel suitable for all phototypes (I-VI) and any skin type.

Contraindications:

Do not use in case of skin diseases and/or infection (dermatitis, herpes).

Do not use if the patient is sensitive to one or more components; we recommend testing the product on an area of the body (e.g. forearm skin, skin behind the ear) to check how the skin of each patient reacts.

Do not use during pregnancy or breastfeeding.

Do not use on children under 12 years of age.

Warnings:

Medical device for external use, to be used by medical professionals only. Avoid contact with eyes

and mucous membranes. Use carefully if the patient has keloids.

Do not use the product past the expiry date shown on the package or if said package is damaged, opened or stored improperly.

Only use BioAgePeel in compliance with these instructions for use.

Avoid exposing the treated area to direct sunlight or UV lamps until any redness caused by the treatment has disappeared.

Side effects:

- Slight reddening of the treated areas, which goes away within 24-48 hours

- After the peel, the skin may become drier. Discontinue use in case of sensitivity or irritation.

Protocol of use:

Before starting treatment, the doctor must inform the patient about the composition, warnings, contraindications and potential adverse reactions associated with this device.

The treatment should be carried out every 7-8 days for at least 4-6 sessions according to medical advice and depending on the purpose of the treatment.

How to apply:

Before starting treatment, cleanse the area to be treated (face, neck, cleavage or other small areas of the body). The doctor is advised to use gloves for protection against the acidic nature of the device

and to protect the patient's mucous membranes, eye and periocular area, which must not come into contact with the product. Using a measuring cap, take 2-5 ml of product; the amount may vary

depending on the area of the body and the skin lesion to be treated. Pour the product into a clean container, then apply a thin layer with a brush or single-use gauze, carefully avoiding mucous membranes, and the eye and periocular area.

Recommended leave-on time: 4-6 minutes, the amount of time may vary depending on the type of skin and the characteristics of the skin problem to be treated. The patient may feel a slight tingling sensation during application.

Remove any product residue with a gauze soaked in water and thoroughly cleanse the treated area of skin.

Apply an emollient cream at the end of the treatment.

The skin will be soft and smooth, with a slight redness that goes away within a few hours.

Packaging:

One pack of BioAgePeel contains:

- five 5 ml ampoules

• Instructions for use to be read carefully before using the device

Storage:

Make sure the product is well closed and store at a temperature between 5°C and 25°C.

Avoid direct contact with sunlight.

Use within one month of opening.

If the colour goes darker, the safety and effectiveness of the device will not be altered or affected.

Do not dispose of in wastewater, since it is hazardous to the aquatic environment. Any product

residue must be disposed of in the appropriate pharmaceutical waste bin, in the original container.

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